



Maintenance and Cleaning Guide

In this guide

- Cleaning Frequency
- Tip and tip guide replacement
- Replacing the tip
- Inserting the tip into the tip guide
- Reattaching the tip guide
- Battery Information
- Charging the battery
- Replacing the battery
- Changing the battery

Cleaning Frequency

During Use: Occasionally rinse the tip and tip guide under the tap while the Piccolo is operating.

After Each Use: With the Piccolo still running and the tip fully extended, rinse both the tip and the tip guide under running water.

Deep Clean: Once a month give your Piccolo a deep clean. With the Piccolo running, remove the tip guide by twisting it anticlockwise. The tip will remain attached. Turn off the Piccolo and remove the tip. Soak both the tip and tip guide together with the Piccolo body, tip end facing down, up to the blue ring, overnight in a glass of properly mixed denture cleaning solution. Rinse all parts thoroughly and reassemble before the next use.

Tip and Tip Guide Replacement

Tip Replacement: Tips are reusable but should be replaced monthly.

Tip Guide Replacement: Replace the tip guide if it becomes dirty or damaged

Replacing the Tip

1. Turn on the Piccolo.
2. Remove the tip guide with a half twist anticlockwise. The tip will remain attached to the body.
3. Turn off the Piccolo and remove the tip.
4. Insert a new tip into the tip guide, pointy end first, ensuring the bottom of the tip is level with the bottom of the guide.
5. Reconnect the tip guide to the Piccolo body.

Inserting the Tip into the Tip Guide

Thread the tip into the guide, pointy end first. Avoid exiting through the side openings by gently rotating the tip during insertion. Ensure the bottom of the tip is level with the bottom of the guide.



Maintenance and Cleaning Guide

Reattaching the Tip Guide

Ensure a tip is inserted, pointy end first. Gently push the tip into the top of the Piccolo body and secure it with a clockwise half twist until firmly in place.

Battery information

The Piccolo uses an AA battery with a built-in USB-C port for recharging, eliminating the need for a separate charger.

Charging the battery

Charge using a standard USB-C cable connected to a wall adapter or computer. The battery is fully charged when the flashing green light stops flashing.

Replacing the battery

The supplied battery should last for a very long time. If replacing the battery, we recommend changing the battery with the same re-chargeable USBC battery type. If this type is not available, we recommend Energizer Ultimate Lithium AA batteries, available at most supermarkets. Or a standard AA battery in the event of a USBC battery type or a Lithium AA battery not being available.

Changing the battery

1. Remove the grip
 - Hold the white neck of the Piccolo body.
 - Using your other hand, grip the gold band and twist anticlockwise to remove the grip
2. Insert the battery with the positive end facing the tip and the USB - C Connector exposed.
3. Reattach the grip
 - Hold the white neck of the Piccolo body.
 - Using your other hand, grip the gold band and twist by twisting clockwise until secure
4. Turn the product on, squeeze and release the grip ensuring that the grip reinflates. In the case of the grip not re-inflating, remove and reattach the grip.